# APP MIAVITAD

## GUIDA D'USO COMPLETA

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### 3. FIRST LAUNCH, REGISTRATION AND LOGIN

- **REGISTRATION (SIGN UP)**  
If you do not have an account, register using an email address and password:
  - **Username**
  - **Email**
  - **Password and Confirm Password** (eye icon to show/hide)
  - Check **I accept the terms and conditions** (dedicated link)
  - **Sign Up** button

Registration is mandatory in order to use the app.

- **LOGIN**

- Enter your **email and password** to log in.
- If you do not remember your password, tap **Forgot Password**: enter your email address and you will receive an email with a link to **reset your password**.
- Enter your email and password.
- If you have forgotten your password, use **Forgot Password** (see section 10).

# MIAVITAD APP

## PROFESSIONAL USER GUIDE

Welcome to **MiaVitaD**! The Nature Light application designed to enable the tracking of sessions performed with lamps equipped with **VITA D** technology and the monitoring of estimated and absorbed **International Units (IU) of Vitamin D**, calculated based on the user's **phototype**.

### 1. WHAT IS MIAVITAD AND WHAT IS IT USED FOR

- **Main function:** connecting the smartphone to lamps equipped with **VITA D** technology in order to start, monitor, and save exposure sessions.
- **What it measures:** the estimated and **accumulated International Units (IU)** of Vitamin D during each session, based on the selected **phototype**.
- **Where to find your data:** in the **Information** → **Sessions** section, with session lists and graphs.

Tip: Always keep the app updated and keep the lamp charged. The session screen also displays the lamp's **battery indicator**.

### 2. SYSTEM REQUIREMENTS

- iOS or Android smartphone with an active internet **connection**.
- Internet is required to: **create an account**, **log in**, **save** sessions to the cloud, **view** graphs and history.
- Lamp equipped with **VITA D** technology physically switched on to start the counting.

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## 4. APP NAVIGATION

Navigation is **simple** and based on the icons at the top:

- **Back arrow** < in the top left to go back.
- **Menu/icons** at the top to access sections (Devices, Information, Settings, etc.).
- The pages are few and straightforward, designed to quickly access your sessions and data.

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## 5. PAIRING A VITA D LAMP

- Open **Devices**
- **Enable Bluetooth** if prompted.
- Tap **Refresh** to scan for nearby lamps
- The lamp will appear with:
  - Device name** e.g. VITA-D-93F0-3726
  - Estimate distance** (e.g. ~0.54 m)
  - Status** (e.g. Available)
  - Signal strength** (dBm)

- Once connected, the app automatically redirects to the **Session Screen**.

Note: Leaving the session screen **disconnects** the lamp and automatically **saves** the active session.

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## 6. SESSION SCREEN

Starting and stopping a session

- **To start:** connect the lamp and **switch it on physically**. The app will **automatically** start the timer according to the selected phototype.
- **To stop and save:**
  - **Turn off the lamp, or**
  - **Exit** the session screen (back).
  - In both cases, the session is **saved online** and the device **disconnects**.

• Note: an **Internet connection** is required to save the session and view it in the history.

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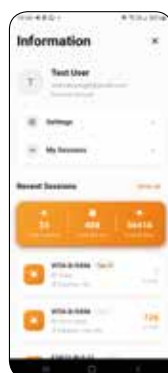
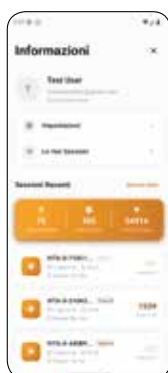
## 7. HOME → INFORMATION

The **Information** page displays your profile (name, email) and provides two shortcuts:

- **Settings**
- **Sessions**

"Recent Sessions" panel

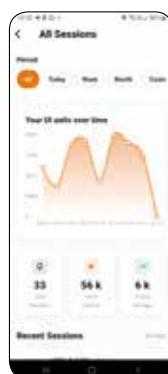
- **Total sessions:** total number of sessions (e.g., 75)
- **Total minutes:** total minutes (e.g., 465)
- **Total IU:** total International Units of Vitamin D (e.g., 54,916)
- List of recent sessions including: date/time, **phototype** (Type I/III badge), duration, and IU.

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## 8. SESSION HISTORY AND GRAPHS

In **Information** → **Sessions** the user can view:

- Total sessions (e.g. 75)
- Total accumulated IU (e.g. 55k units)
- Average IU/day
- **Chronological list of sessions**
- Period-based analytics (Daily/Monthly/Custom)

Recommendations: use **Month** for routine monitoring, **Custom** for comparing specific intervals.

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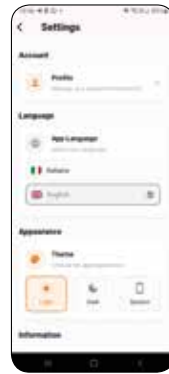
## 9.SETTINGS

Open **Information** → **Settings**.

- **Language**
- **App Language:** Italian or English

**Theme**

- **Light, Dark,** or System (adapts the theme to the phone's settings). The change is applied immediately.

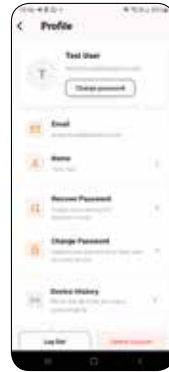
**Contacts & Legal**

- **Contact Us:** phone, email, physical address
- **Terms and Conditions:** view the terms

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## PROFILE

- Go to **Profile** to manage your personal data:
- **Email** (view only)
- **Name** (editable)
- **Forgot Password** ("Forgot Password" procedure, see section 10)
- **Change Password** (change your password from the app)
- **Device History** (list of used devices )
- Log Out
- Delete Account (requires password re-entry)

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## 10. PASSWORD & ACCOUNT MANAGEMENT

**Forgot Password**

- Accessible from login or **Profile**
- Enter account **email**.
- Submit request
- Follow the reset **link** received via email

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**Change password**

- In **Profile** → **Change password**.
- Enter current **password and new password**.
- Confirm to save.

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Tip: Use a long password (at least 8–12 characters, with upper/lowercase letters, numbers, and symbols).

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**Delete Account**

- In **Profile** → **Delete account**.
- For security reasons, you will be asked to **re-enter your password**.
- The operation will not proceed without confirmation.

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## 11. BEST PRACTICES

- Select the correct phototype before switching on the lamp.
- Do not cover sensors; maintain the manufacturer-recommended operating distance.
- Check battery level before beginning a session.
- To save a session, turn off the lamp or exit the session screen
- Review exposure charts regularly to maintain safe Vitamin D accumulation levels.

## 12. TROUBLESHOOTING (FAQ)

Lamp not detected by your device.

- Ensure Bluetooth is active.
- Verify permissions.
- Move closer to the lamp and rescan.
- Power cycle the lamp and try again.

IU counter not starting.

- Ensure lamp is physically on.
- Confirm phototype selection.
- Remain on the session screen (leaving disconnects Bluetooth).

Session not saving.

- Check internet connectivity.
- Verify under Information → Sessions.

New smartphone

- Log in with the same email and reconnect the lamp via Devices.

## 13. PRIVACY AND SECURITY

- User credentials are encrypted and securely managed. All sensitive actions require password confirmation.

### Thank you for choosing Nature Light.

For technical support or feedback, please contact us via Settings → Contact Us.

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© NATURE LIGHT srl – P. IVA 04349080244 – 27/11/2025